

Table of contents

\USER

Brain

HARP_Skyra_VE11C

HARP_20190617

AAHead_Scout
AC-PC_setup
SEField1_AP
BOLD_REST1_AP
SEField1_PA
BOLD_REST1_PA
T1_MPR
T2_SPC
ASL_ADNI
QSM_3D
DWI_AP
DWI_PA
SEField2_AP
BOLD_REST2_AP
SEField2_PA
BOLD_REST2_PA
SEField3_AP
BOLD_REST3_AP
SEField3_PA
BOLD_REST3_PA
SEField4_AP
SEField4_PA
BOLD_EMOTION_PA
SEField5_AP
SEField5_PA
BOLD_CARIT_PA

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\AAHead_Scout

TA: 0:14 PM: REF Voxel size: 1.6×1.6×1.6 mmPAT: 3 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
TE	1.37 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3.15 ms
TE	1.37 ms
Flip angle	8 deg

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Resolution - Common

FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
Base resolution	160
Phase resolution	100 %
Slice resolution	69 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Trajectory	Cartesian

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	24

Resolution - iPAT

Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H

System - Miscellaneous

Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Flip angle	8 deg
Measurements	1
Time to center	6.2 s

Inline - Inline

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle	8 deg
Measurements	1
Contrasts	1
TR	3.15 ms
TE	1.37 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Asymmetric echo	Weak
Contrasts	1
Multi-slice mode	Sequential
Bandwidth	540 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\AC-PC_setup

TA: 6.1 s PM: ISO Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField1_AP

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST1_AP

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField1_PA

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST1_PA

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\T1_MPR

TA: 5:22 PM: REF Voxel size: 0.8×0.8×0.8 mmPAT: 2 Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	7.1 %
Slices per slab	224
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2500.0 ms
TE	2.18 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2500.0 ms
TE	2.18 ms
Magn. preparation	Non-sel. IR
T1	1000 ms
Flip angle	8 deg
Fat suppr.	Water excit. fast
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	6/8
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	7.1 %
Slices per slab	224
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2500.0 ms
Multi-slice mode	Single shot
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	3.02 deg
Initial Orientation	Sagittal

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Sagittal
Rotation	3.02 deg
A >> P	240 mm
F >> H	256 mm
R >> L	180 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2500.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI	1000 ms
Fat suppr.	Water excit. fast
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle	8 deg
Measurements	1
TR	2500.0 ms
TE	2.18 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	7.9 ms
Bandwidth	220 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
RF spoiling	On
Incr. Gradient spoiling	Off
Turbo factor	240

Sequence - Assistant

Mode	Off
------	-----

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\T2_SPC

TA: 5:31 PM: FIX Voxel size: 0.8×0.8×0.8 mmPAT: 2 Rel. SNR: 1.00 : spc

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	7.1 %
Slices per slab	224
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
TE	564 ms
Averages	1.0
Concatenations	1
Filter	Prescan Normalize, Image Filter
Coil elements	HEA;HEP

Contrast - Common

TR	3200 ms
TE	564 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None
Blood suppr.	Off
Restore magn.	Off

Contrast - Dynamic

Averages	1.0
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	On
Intensity	Sharp
Edge Enhancement	3
Smoothing	3
Unfiltered images	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	7.1 %
Slices per slab	224
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	3.02 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	Off
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H

Geometry - Tim Planning Suite

Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Sagittal
Rotation	3.02 deg
A >> P	240 mm
F >> H	256 mm
R >> L	180 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	3200 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
---------------	-----

Physio - PACE

Concatenations	1
----------------	---

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	On
Reordering	Linear
Flow comp.	No
Echo spacing	3.86 ms
Adiabatic-mode	Off
Bandwidth	679 Hz/Px

Sequence - Part 2

Echo train duration	1166 ms
RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
Flip angle mode	T2 var
Turbo factor	314

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\ASL_ADNI

TA: 2:45 PM: FIX Voxel size: 3.0x3.0x4.0 mmRel. SNR: 1.00 : tgse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	10 %
Slice oversampling	20.0 %
Slices per slab	40
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	4.00 mm
TR	5000 ms
TE	16.02 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	5000 ms
TE	16.02 ms
Flip angle	180 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Contrast - ASL

Perfusion mode	FAIR QII
Suppression Mode	GRAY-WHITE
Bolus Duration	800 ms
Inversion Time	2000 ms
Averaging mode	CONSTANT
Inversion Array Size	1

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	4.00 mm

Resolution - Common

Base resolution	64
Phase resolution	98 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice oversampling	20.0 %
Slices per slab	40
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	4.00 mm
TR	5000 ms
Multi-slice mode	Interleaved
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T

System - Miscellaneous

Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	192 mm
R >> L	192 mm
F >> H	160 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	5000 ms
Concatenations	1
Segments	16

Sequence - Part 1

Introduction	Off
Dimension	3D
Reordering	Centric
Multi-slice mode	Interleaved
Echo spacing	0.51 ms
Bandwidth	2604 Hz/Px

Sequence - Part 2

EPI factor	21
Segments	16
RF pulse type	Normal
Gradient mode	Fast
Turbo factor	12

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\QSM_3D

TA: 5:03 PM: FIX Voxel size: 0.9×0.9×1.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	Head > Brain
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	128
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	44.0 ms
TE 1	3.60 ms
TE 2	9.51 ms
TE 3	15.43 ms
TE 4	21.34 ms
TE 5	27.26 ms
TE 6	33.17 ms
TE 7	39.09 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	44.0 ms
TE 1	3.60 ms
TE 2	9.51 ms
TE 3	15.43 ms
TE 4	21.34 ms
TE 5	27.26 ms
TE 6	33.17 ms
TE 7	39.09 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magn./Phase

Contrast - Dynamic

Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	50 %
Phase partial Fourier	6/8
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	0.0 %
Slices per slab	128
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	44.0 ms
Multi-slice mode	Interleaved
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm

Geometry - AutoAlign

Initial Rotation	90.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	240 mm
A >> P	240 mm
F >> H	128 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	44.0 ms
Concatenations	1

Physio - Signal1

Segments	1
----------	---

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	240 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Noise threshold	25
Save original images	On
MapIt	T2* map
Flip angle	15 deg
Measurements	1
Contrasts	7
TR	44.0 ms
TE 1	3.60 ms
TE 2	9.51 ms
TE 3	15.43 ms
TE 4	21.34 ms
TE 5	27.26 ms
TE 6	33.17 ms
TE 7	39.09 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Off

Sequence - Part 1

Contrasts	7
Flow comp. 1	No
Readout mode	Monopolar
Multi-slice mode	Interleaved
Bandwidth 1	240 Hz/Px
Bandwidth 2	240 Hz/Px
Bandwidth 3	240 Hz/Px
Bandwidth 4	240 Hz/Px
Bandwidth 5	240 Hz/Px
Bandwidth 6	240 Hz/Px
Bandwidth 7	240 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\DWI_AP

TA: 4:50 PM: FIX Voxel size: 1.7×1.7×1.7 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	84
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
TR	3600 ms
TE	89.00 ms
Multi-band accel. factor	3
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3600 ms
TE	89.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	None
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
Base resolution	120
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2

Resolution - iPAT

Ref. lines PE	36
Reference scan mode	GRE

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	84
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
TR	3600 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L

System - Miscellaneous

Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	204 mm
R >> L	204 mm
F >> H	143 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	3600 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	67
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	67
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.74 ms
Bandwidth	1544 Hz/Px

Sequence - Part 2

EPI factor	120
Gradient mode	Fast*
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	5120 us
Refocus pulse duration	10240 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	On
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
GRE iPAT ref. FA	12.0 deg
Physio recording	Off

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\DWI_PA

TA: 4:54 PM: FIX Voxel size: 1.7×1.7×1.7 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	84
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
TR	3600 ms
TE	89.00 ms
Multi-band accel. factor	3
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3600 ms
TE	89.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	None
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
Base resolution	120
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2

Resolution - iPAT

Ref. lines PE	36
Reference scan mode	GRE

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	84
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	204 mm
FoV phase	100.0 %
Slice thickness	1.70 mm
TR	3600 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L

System - Miscellaneous

Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	204 mm
R >> L	204 mm
F >> H	143 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	3600 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	68
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	2000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.74 ms
Bandwidth	1544 Hz/Px

Sequence - Part 2

EPI factor	120
Gradient mode	Fast*
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	5120 us
Refocus pulse duration	10240 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	On
Invert RO/PE polarity	On
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
GRE iPAT ref. FA	12.0 deg
Physio recording	Off

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField2_AP

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST2_AP

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField2_PA

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST2_PA

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField3_AP

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST3_AP

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField3_PA

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_REST3_PA

TA: 5:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	375
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField4_AP

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField4_PA

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_EMOTION_PA

TA: 4:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField5_AP

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\SEField5_PA

TA: 6.1 s PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
TE	60.00 ms
Multi-band accel. factor	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	6100 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	6100 ms
Multi-band accel. factor	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
RF pulse type	Normal

Sequence - Part 2

Gradient mode	Fast*
Excitation	Standard

Sequence - Special

SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\Brain\HARP_Skyra_VE11C\HARP_20190617\BOLD_CARIT_PA

TA: 4:08 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : efpid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
TE	34.40 ms
Multi-band accel. factor	6
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	34.40 ms
MTC	Off
Magn. preparation	None
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
Base resolution	86
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	206 mm
FoV phase	100.0 %
Slice thickness	2.40 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	6

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
--------------	----------

System - Adjustments

B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	206 mm
R >> L	206 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.251219 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	800 ms
Multi-band accel. factor	6

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.63 ms
Bandwidth	2076 Hz/Px

Sequence - Part 2

EPI factor	86
Gradient mode	Fast*

Sequence - Part 2

Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	6600 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard